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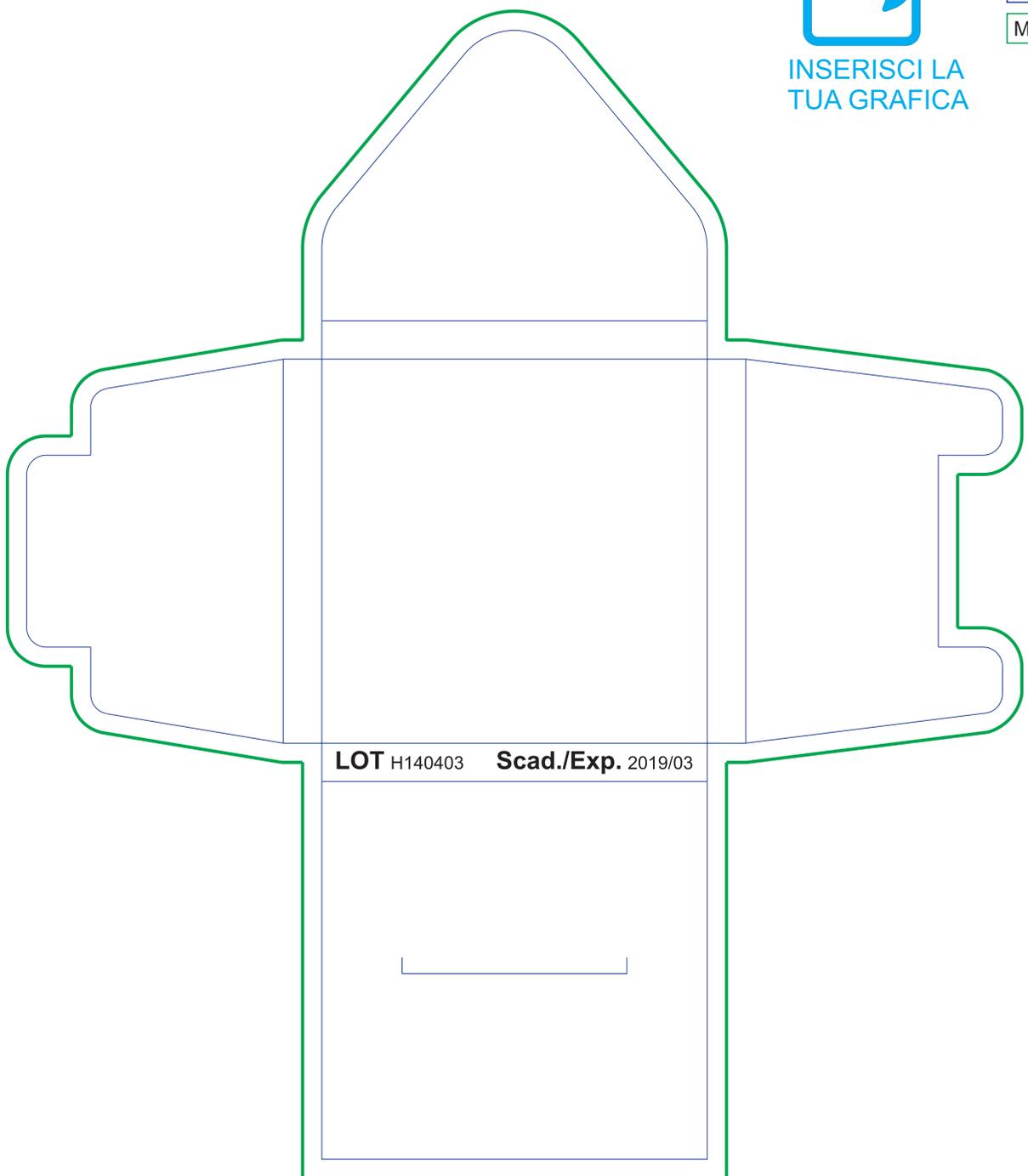
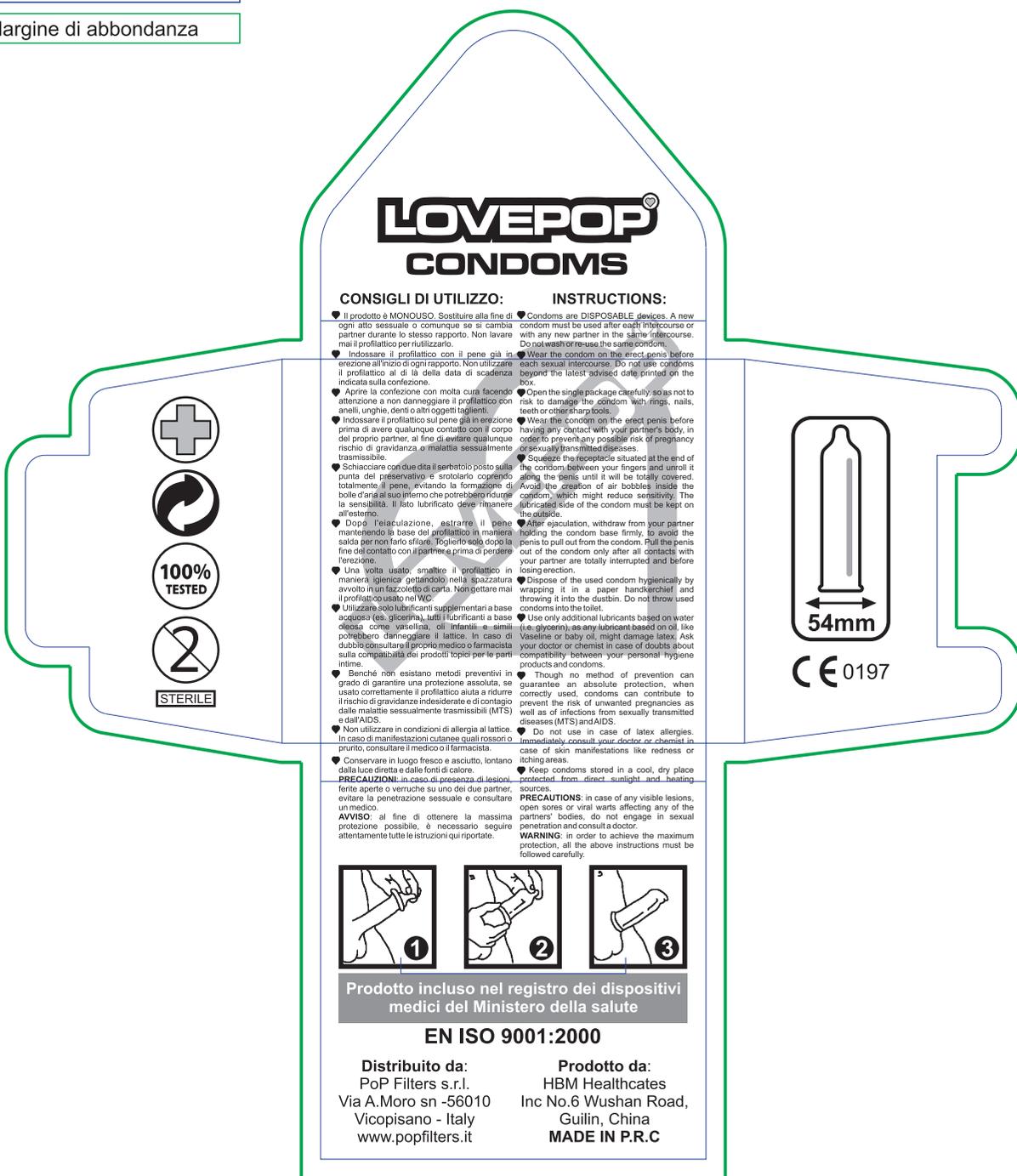
Fustella

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LOT H140403 Scad./Exp. 2019/03

ESTERNO


LOVEPOP
CONDOMS

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INSTRUCTIONS:

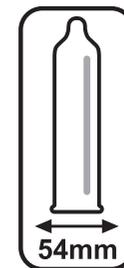- Condoms are DISPOSABLE devices. A new condom must be used after each intercourse or with any new partner in the same intercourse. Do not wash or re-use the same condom.
- Wear the condom on the erect penis before each sexual intercourse. Do not use condoms beyond the latest advised date printed on the box.
- Open the single package carefully so as not to risk to damage the condom with rings, nails, teeth or other sharp objects.
- Wear the condom on the erect penis before having any contact with your partner's body, in order to prevent any possible risk of pregnancy or sexually transmitted diseases.
- Squeeze the receptacle situated at the end of the condom between your fingers and unroll it along the penis until it will be totally covered. Avoid the creation of air bubbles inside the condom, which might reduce sensitivity. The lubricated side of the condom must be kept on the outside.
- After ejaculation, withdraw from your partner holding the condom base firmly, to avoid the penis to pull out from the condom. Pull the penis out of the condom only after all contacts with your partner are totally interrupted and before losing erection.
- Dispose of the used condom hygienically by wrapping it in a paper handkerchief and throwing it into the dustbin. Do not throw used condoms into the toilet.
- Use only additional lubricants based on water (e.g. glycerin), as any lubricant based on oil, like Vaseline or baby oil, might damage latex. Ask your doctor or chemist in case of doubts about compatibility between your personal hygiene products and condoms.
- Though no method of prevention can guarantee an absolute protection, when correctly used, condoms can contribute to prevent the risk of unwanted pregnancies as well as of infections from sexually transmitted diseases (MTS) and AIDS.
- Do not use in case of latex allergies. Immediately consult your doctor or chemist in case of skin manifestations like redness or itching areas.
- Keep condoms stored in a cool, dry place protected from direct sunlight and heating sources.

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Prodotto da:
HBM Healthcates
Inc No.6 Wushan Road,
Guilin, China
MADE IN P.R.C

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